

# AliMed® Patient Roller

## Instructions for Use and Care

MD


### Intended Use

The AliMed Patient Rollers are used to transfer patients across different surfaces with minimal disruption to the status of the patient.

### Instructions for Use

Reusable.

1. Lock stretcher against patient bed/O.R. table. Tilt patient.
2. Position the Patient Roller under patient. Lay patient back down.
3. Push the Patient Roller onto the stretcher or other transfer surface.
4. Remove the Patient Roller from beneath the patient.

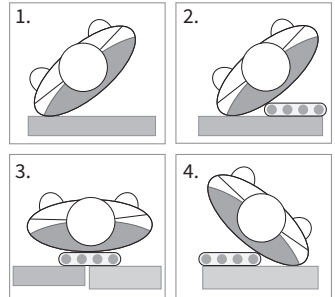

### Contraindications/Warnings



- This is not a spine or backboard. Use extreme caution to avoid spine and neck injuries.
- Ensure proper transfer and lifting techniques are utilized to minimize the risk of user injury.
- Consult your institution's guidelines for assessing the risk of each individual patient.
- Report any serious incident to AliMed and the appropriate regulatory body.

### Care and Cleaning

Do not use cleaners with the active ingredient of glutaraldehyde as this may damage and shorten the cover life. Approved cleaners: Soap and water, bleach solution of 10% or less. Clean Patient Roller after each patient is successfully transferred. Using improper cleaning solutions or methods can result in damage to the product or irritation to the patient.

### Materials



Component A: Aluminum rollers and frame  
Component B: Vinyl outer covering  
Not made with natural rubber latex.



**AliMed® Inc.**

297 High St., Dedham, MA 02026 USA  
PHONE 800.225.2610 • FAX 800.437.2966

**AliMed.com**

©2024 AliMed® Inc. All rights reserved.  
IFU-0098R01 ECO1662  
8165REV0524

Storage/Disposal

Store in a clean, dry environment protected from direct sunlight. Storage at ambient room temperature is advised. Dispose of in accordance with existing federal, state, and local environmental control laws.

Symbol Glossary



Medical Device



Device is non-sterile



Not made with Natural Rubber Latex



Caution



Date of Manufacture



Manufacturer



Manufacturer's catalog number

Item Numbers

<b>REF</b>	<b>2970012619</b>	<b>2970018248</b>	<b>2970018247</b>	<b>2970011822</b>	<b>2970017439</b>	<b>2970017438</b>	<b>2970011819</b>	<b>2970011820</b>
(Former Item #)	(9-254)	(9-729)	(9-728)	(9-209)	(9-478)	(9-477)	(920889)	(920890)
<b>REF</b>	<b>2970012619</b>							
(Former Item #)	(920891)							

Warranty

ALIMED warrants to Purchaser that the Product(s) sold hereunder, is at the time of shipment to Purchaser from ALIMED, free from defects in material and workmanship. As Purchaser’s sole exclusive remedy under this warranty ALIMED agrees either to repair or replace, at ALIMED’s sole option and free of charge to Purchaser, any part or parts of such Product(s) which under proper and normal conditions of use prove to be defective within six months from the date of shipment. Visit **AliMed.com/terms** for full warranty info.

California Proposition 65 Statement:  
**65** **WARNING:** This product can expose you to chemicals including Di(2-Ethylhexyl) Phthalate (DEHP), which is known to the State of California to cause cancer and birth defects or other reproductive harm. For more information go to <https://www.p65warnings.ca.gov>.



**AliMed® Inc.**

297 High St., Dedham, MA 02026 USA  
PHONE 800.225.2610 • FAX 800.437.2966

**AliMed.com**  
©2024 AliMed® Inc. All rights reserved.  
IFU-0098R01 ECO1662  
8165REV0524